

BD Vacutainer® Plus Plastic Citrate Tube

BD Vacutainer® Plus Plastic Citrate Tube Draw Volume Guide


*Sufficient volume achieved if blood drawn falls above minimum fill indicator.
For blood transfer, **do not** fill above illustrated dashed maximum line.*

Note: The quantity of blood drawn into evacuated tubes varies with altitude, ambient temperature, barometric pressure, tube age, venous pressure and filling technique.

**2.7 mL
Draw
Tube**

13 mm x 75 mm
Full Draw

Ref. # 363083

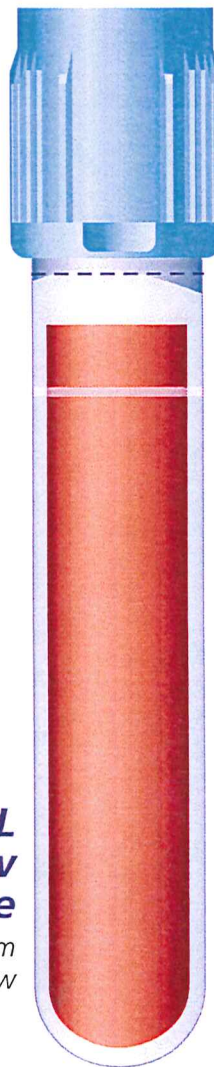


Maximum Fill*

Minimum Fill
Indicator

**Now with
Minimum Fill
Indicator**

*Represents minimum
volume of blood
required for
appropriate analysis*

Ref. # 363080

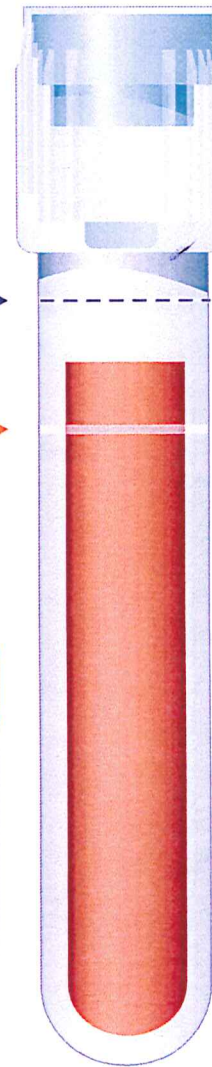

**1.8 mL
Draw
Tube**

13 mm x 75 mm
Full Draw

*According to CLSI guideline,
Dec. 2003, Doc. H1-A5, Vol. 23, No. 33.