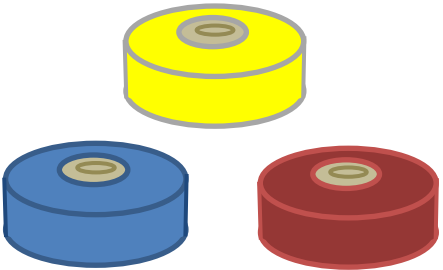
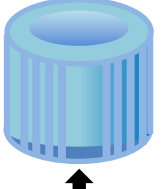
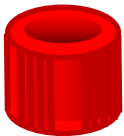

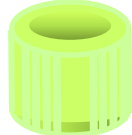
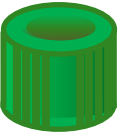








**Quick Reference Guide:**

For more information please visit:  
[lab.parkview.com](http://lab.parkview.com) or call 260-266-1500

**Order of Draw / Tube Description**

Tube Caps	Additive	Common Testing
	Pediatric (yellow) Aerobic (Blue) Anaerobic (Maroon)	<ul style="list-style-type: none"> <li>Blood Culture</li> </ul>
 ↑ Blue Top	Sodium Citrate / Whole Blood	<ul style="list-style-type: none"> <li>Prothrombin Time (PT/INR)</li> <li>Activated Partial Thromboplastin (APPT/PPT)</li> <li>Fibrinogen</li> <li>D-Dimer</li> </ul>
 ← Red Top	Clot activator	<ul style="list-style-type: none"> <li>Drug Levels (Non-SST)</li> </ul>
Gold Top (SST) → 	Serum Separator Tube (SST)	<ul style="list-style-type: none"> <li>Vitamin D</li> <li>PSA</li> </ul>
 ← Light Green (PST)	Plasma Separator Tube (PST)	<ul style="list-style-type: none"> <li>Renal function profile</li> <li>Iron profile</li> <li>TSH</li> <li>BMP/CMP/Chem6</li> <li>Lipid / Liver profile</li> </ul>
Dark Green → 	Sodium/Lithium Heparin	<ul style="list-style-type: none"> <li>Na Hep – Chromosome Testing</li> </ul>
 ← Lavender Top	EDTA – Whole Blood	<ul style="list-style-type: none"> <li>CBC w/Diff</li> <li>CBC wo/Diff</li> <li>Glycohemoglobin (A1C)</li> <li>Sed. Rate</li> <li>RBC, Folate</li> </ul>
Pink Top → 	K <sub>2</sub> (Potassium)	<ul style="list-style-type: none"> <li>Blood Bank</li> </ul>
 Grey Top	Sodium Fluoride Potassium Oxalate/ Whole Blood	<ul style="list-style-type: none"> <li>Lactic Acid</li> </ul>
 Navy Top	K <sub>2</sub> (Potassium) EDTA/Whole Blood	<ul style="list-style-type: none"> <li>Trace Element</li> <li>Lead Testing</li> </ul>

**Contact Information**

Parkview Health Laboratories: (260)266-1500

➤ Client Response Center – option 1